

## Application of Colour Liquid Aquarell Bio-Pigment



The colour pigments, which are added to the Colour Liquid Prettau® Aquarell in order to easily visualise the application of liquid to the zirconia, may degrade over time (e.g. if stored in bright light). By adding Colour Liquid Aquarell Bio-Pigment, these colour pigments can be enhanced. The colours can be made as intensive as you like.

### Colour assignment:

Colour Liquid Prettau® Aquarell	A1 – D4	Cervical A-D	Incisal Blue	Incisal Grey	Incisal Violet	Orange 1	Brown 2	Tissue A-C
Colour Liquid Aquarell Bio-Pigment	A1 – D4	Orange	Blue	Grey	Violet	Orange	Brown	Tissue

### 1 INITIAL SITUATION



The Colour Liquid Aquarell Bio-Pigment is used for enhancing the Colour Liquid Prettau® Aquarell and will burn out in the furnace. Make sure the colours are assigned correctly during the application!

### 2 PREPARE BIO-PIGMENT



Shake the Colour Liquid Aquarell Bio-Pigment well before using it.

### 3 OPEN



Open both bottles.

### 4 ADD BIO-PIGMENT



Add Colour Liquid Aquarell Bio-Pigment to Colour Liquid Prettau® Aquarell (1-2 drops Bio-Pigment per cm of liquids in the bottle). To achieve an even more intensive colour, some more Bio-Pigment can be added.

### 5 MIX



Shake the bottle with the Colour Liquid Prettau® Aquarell well to mix the liquids.

### 6 DYEING



The enhanced Colour Liquid Prettau® Aquarell can now be used!

