

Zirkonzahn[®]

Human Zirconium Technology



PRETTAU ZIRCONIA

Prettau Zirconia

Prettau frame-work can be milled to 'dentine-build-up' dimensions to be veneered with enamel porcelain only. The advantage: Veneer-firing only a thin enamel layer is a substantial time saver. Overall frame strength increases and the risk of porcelain chipping pretty much eliminated.

Milled Prettau framework is custom coloured using Prettau Colour Liquids prior to sintering. After sintering a dentin wash-bake is applied and the case finished with a thin enamel layer of 0.3 to 0.5mm thickness.

Even with such time-saving technique, the excellent translucency of Prettau zirconia delivers the same aesthetic result as a conventionally veneered restoration.



Prettau Zirconia

Aldo Zilio, Italy



Dentine framestructure made from 100% prettau zirconia



Zirkonzahn®



HUMAN ZIRCONIUM TECHNOLOGY

ZIRKONZAHN USA Inc. - Tel 1 800 989 8931 - Tel 678 441 9419 - infousa@zirkonzahn.com

ZIRKONZAHN World Wide - Tel +39 0474 066 680 - info@zirkonzahn.com - www.zirkonzahn.com



Milled with the Zirkonzahn Milling System



WEAA5405=