



PRETTAU[®] 2 DISPERSIVE[®]

The Zirkonzahn Culture

Zirkonzahn

PRETTAU® BRIDGES MADE OF PRETTAU® 2 DISPERSIVE® ON ANODISED TITANIUM BARS


Initial situation:


Data material: Patient request: Planned restoration: Realisation: _____ Male patient, middle age, edentulous upper and lower jaw, with four implants each; temporary restoration with two removable complete dentures Photos, master models with implants, situ scans of the complete dentures Highly aesthetic, long-lasting, fixed restoration; new carefree smile Full-arch Prettau[®] Bridges in Prettau[®] 2 Dispersive[®], with improved occlusal scheme

- Transfer patient data into virtual articulator
- Realisation of new resin prototypes with improved occlusion in Zirkonzahn. Modellier based on situ scans and individual tooth libraries
- Mill resin prototypes and veneer gingival aspect
- After try-in: situ scan of prototypes to transition to bar-supported zirconia restoration
- Design, mill and re-scan titanium bars
- Adapt virtual waxup to scanned bars
- Mill arches in Prettau[®] 2 Dispersive[®] with the Zirkonzahn milling unit
- Characterisation of the gingival texture, colour accentuation and sintering of the structures
- Post-process monolithic bridges with ICE Stains 3D by Enrico Steger; layer gingival aspect with ceramics
 - Control and anodisation of the titanium bars; cementation into zirconia structures

Result:

Optimal restoration for the patient according to aesthetic and functional aspects

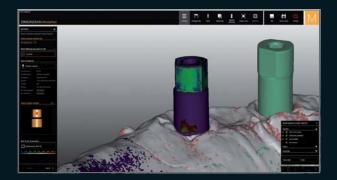

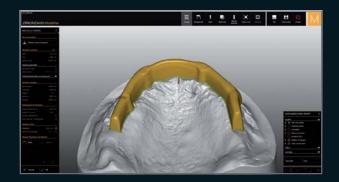





100 % MONOLITHIC DESIGN, VENEERED ONLY IN THE GINGIVAL AREA

DT Federico Presicci, Dental Laboratory Antonio Lazetera, Savona, Italy Dr. Nicola De Angelis & Dr. Marco De Lorenzi, Acqui Terme, Italy







Zirkonzahn

PRETTAU® 2 DISPERSIVE®



NEW! PRETTAU® 2 DISPERSIVE®

- Pre-coloured zirconia with a natural colour gradient, optimal flexural strength and particularly high translucency
- No limitations! Ideal for monolithic full arch bridges, but also for single crowns, inlays, onlays, veneers, bars and multi-unit bridges (reduced or monolithic)
- No ceramic chipping (thanks to monolithic design), no abrasion of the antagonist
- Can be characterised individually for each patient with Colour Liquid Prettau[®] 2 Aquarell Intensive, ICE Ceramics, Fresco Ceramics and ICE Stains 3D by Enrico Steger



HUMAN ZIRCONIUM TECHNOLOGY

Zirkonzahn Worldwide – Tel +39 0474 066 680 – info@zirkonzahn.com – www.zirkonzahn.com



WEAB2676EN=

This is an international advertisement. Not all products mentioned are available in all countries. The product's area of application may differ according to the country. Please contact your Sales Team for more detailed information. All information is subject to change. Errors and omissions excepted. Version: 12/01/2022