

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
A-Z®	VL	20 – 25 Ncm	10 – 12 Ncm

Alpha-Bio TEC®

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Alpha-Bio TEC®	SPI/DFI/ATDI 3.75	30 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Astra Tech	Multi Unit 20°	15 Ncm	10 – 12 Ncm
Astra Tech	Multi Unit 45°	15 Ncm	10 – 12 Ncm
Astra Tech	OsseoSpeed 3.0	15 Ncm	10 – 12 Ncm
Astra Tech	OsseoSpeed 3.5 – 4.0	20 Ncm	10 – 12 Ncm
Astra Tech	OsseoSpeed 4.5 – 5.0	25 Ncm	10 – 12 Ncm

Bego

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Bego	Semados®	30 Ncm	10 – 12 Ncm

Biomet 3i™

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Biomet 3i™	Certain® 3.4	20 Ncm	10 – 12 Ncm
Biomet 3i™	Certain® 4.1	20 Ncm	10 – 12 Ncm
Biomet 3i™	Certain® 5.0	20 Ncm	10 – 12 Ncm
Biomet 3i™	Certain® 6.0	20 Ncm	10 – 12 Ncm
Biomet 3i™	OSSEOTITE® 3.4	35 Ncm	10 – 12 Ncm
Biomet 3i™	OSSEOTITE® 4.1	35 Ncm	10 – 12 Ncm
Biomet 3i™	OSSEOTITE® 5.0	35 Ncm	10 – 12 Ncm
Biomet 3i™	OSSEOTITE® 6.0	35 Ncm	10 – 12 Ncm

Bredent SKY

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Bredent SKY	Classic/blueSKY 3.5/4.0/4.5/5.5	25 Ncm	10 – 12 Ncm
Bredent SKY	fast & fixed	20 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
BTI®	Conical Spacer 4.1/5.5	10 Ncm	10 – 12 Ncm
BTI®	Externa® 4.1	20 Ncm	10 – 12 Ncm
BTI®	Externa® 5.5	20 Ncm	10 – 12 Ncm
BTI®	Interna® 4.1	20 Ncm	10 – 12 Ncm
BTI®	Interna® 5.5	20 Ncm	10 – 12 Ncm
BTI®	Multi-IM 3.5/4.1	10 Ncm	10 – 12 Ncm
BTI®	Multi-IM 5.5	10 Ncm	10 – 12 Ncm
BTI®	Multi-IM Angled	10 Ncm	10 – 12 Ncm
BTI®	Tiny® 3.5	20 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Camlog®	Bar Abutments	20 Ncm	10 – 12 Ncm
Camlog®	Conelog® 3.3	20 Ncm	10 – 12 Ncm
Camlog®	Conelog® 3.8	20 Ncm	10 – 12 Ncm
Camlog®	Conelog® 4.3	20 Ncm	10 – 12 Ncm
Camlog®	Conelog® 5.0	20 Ncm	10 – 12 Ncm
Camlog®	J-Type/K-Type 3.3	20 Ncm	10 – 12 Ncm
Camlog®	J-Type/K-Type 3.8	20 Ncm	10 – 12 Ncm
Camlog®	J-Type/K-Type 4.3	20 Ncm	10 – 12 Ncm
Camlog®	J-Type/K-Type 5.0	20 Ncm	10 – 12 Ncm
Camlog®	J-Type/K-Type 6.0	20 Ncm	10 – 12 Ncm

All information is subject to change. Errors and omissions excepted. Version: 22/09/2014

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Camlog®	Vario SR	20 Ncm	10 – 12 Ncm

Dentium

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Dentium	Implantium 3.4	25 – 30 Ncm	10 – 12 Ncm
Dentium	Implantium 4.3	25 – 30 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Dyna®	Octalock®/Helix® 3.6 – 4.0	35 Ncm	10 – 12 Ncm

Friadent DENTSPLY

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Friadent DENTSPLY	Ankylos®	10 – 15 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® 3.4	24 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® 3.8	24 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® 4.5	24 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® 5.5	24 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® MP/TG 3.4	24 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® MP/TG 3.8	24 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® MP/TG 4.5	24 Ncm	10 – 12 Ncm
Friadent DENTSPLY	XiVE® MP/TG 5.5	24 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
ICX®	ICX®-templant® 3.45 – 4.80	25 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
KLOCKNER®	Essential Cone 3.5/4.0/5.0	30 Ncm	10 – 12 Ncm
KLOCKNER®	Essential Cone 4.8	30 Ncm	10 – 12 Ncm
KLOCKNER®	NK2 3.8/4.2	25 Ncm	10 – 12 Ncm
KLOCKNER®	SK2 4.7	25 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
MIS®	Multi Unit Abutment	30 Ncm	10 – 12 Ncm
MIS®	Seven Standard	30 Ncm	10 – 12 Ncm
MIS®	Seven Wide	30 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Moza Grau®	Tapered Screw®	20 Ncm	10 – 12 Ncm

Neoss

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Neoss®	ProActive® 3.25	32 Ncm	10 – 12 Ncm
Neoss®	ProActive® 3.50/4.00	32 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Nobel Biocare®	Brånemark® 3.5	25 Ncm	10 – 12 Ncm
Nobel Biocare®	Brånemark® 4.1	35 Ncm	10 – 12 Ncm
Nobel Biocare®	Brånemark® 5.1	35 Ncm	10 – 12 Ncm
Nobel Biocare®	Multi Unit Abutment NP-RP	15 Ncm	10 – 12 Ncm
Nobel Biocare®	Multi Unit Abutment WP	35 Ncm	10 – 12 Ncm
Nobel Biocare®	Nobel Active® NP	35 Ncm	10 – 12 Ncm
Nobel Biocare®	Nobel Active® RP	35 Ncm	10 – 12 Ncm
Nobel Biocare®	Nobel Active® CC 3.0	15 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Nobel Biocare®	NobelReplace® 3.5	35 Ncm	10 – 12 Ncm
Nobel Biocare®	NobelReplace® 4.3	35 – 45 Ncm	10 – 12 Ncm
Nobel Biocare®	NobelReplace® 5.0	35 – 45 Ncm	10 – 12 Ncm
Nobel Biocare®	NobelReplace® 6.0	35 – 45 Ncm	10 – 12 Ncm

OSSTEM

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
OSSTEM	Convertible Abutment 4.0	30 Ncm	10 – 12 Ncm
OSSTEM	Convertible Abutment 5.0/6.0	30 Ncm	10 – 12 Ncm
OSSTEM	GS/TS 3.5	30 Ncm	10 – 12 Ncm
OSSTEM	GS/TS 4.0/4.5/5.0	30 Ncm	10 – 12 Ncm
OSSTEM	US 3.5	30 Ncm	10 – 12 Ncm
OSSTEM	US 4.1	30 Ncm	10 – 12 Ncm
OSSTEM	US 5.1	30 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
SIC®	SiCace 3.3	20 Ncm	10 – 12 Ncm
SIC®	SiCace 4.2	20 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Straumann®	Bone Level® NC 3.3	35 Ncm	10 – 12 Ncm
Straumann®	Bone Level® RC 4.1	35 Ncm	10 – 12 Ncm
Straumann®	Multi-Base Abutment	35 Ncm	10 – 12 Ncm
Straumann®	NNC	35 Ncm	10 – 12 Ncm
Straumann®	Soft Tissue Level NN 3.5	35 Ncm	10 – 12 Ncm
Straumann®	Soft Tissue Level RN 4.8	35 Ncm	10 – 12 Ncm
Straumann®	Soft Tissue Level WN 6.5	35 Ncm	10 – 12 Ncm

Sweden & Martina

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Sweden & Martina	Kohno/Premium 3.30	30 Ncm	10 – 12 Ncm
Sweden & Martina	Kohno/Premium 3.80	30 Ncm	10 – 12 Ncm
Sweden & Martina	Kohno/Premium 4.25	35 Ncm	10 – 12 Ncm
Sweden & Martina	Kohno/Premium 5.00	35 Ncm	10 – 12 Ncm
Sweden & Martina	Kohno/Premium 6.00	35 Ncm	10 – 12 Ncm
Sweden & Martina	Multi Unit Abutment	15 Ncm	10 – 12 Ncm
Sweden & Martina	Out-Link 3.30	30 Ncm	10 – 12 Ncm
Sweden & Martina	Out-Link 4.10	35 Ncm	10 – 12 Ncm
Sweden & Martina	Out-Link 5.00	35 Ncm	10 – 12 Ncm

Thommen

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Thommen	SPI® 3.5	15 Ncm	10 – 12 Ncm
Thommen	SPI® 4.0	25 Ncm	10 – 12 Ncm
Thommen	SPI® 4.5	25 Ncm	10 – 12 Ncm
Thommen	SPI® 5.0	25 Ncm	10 – 12 Ncm
Thommen	SPI® 6.0	25 Ncm	10 – 12 Ncm

Manufacturer	Type of implant	Recommended torque	Composite resin temporaries
Zimmer®	Tapered Screw-Vent® 3.5	30 Ncm	10 – 12 Ncm
Zimmer®	Tapered Screw-Vent® 4.5	30 Ncm	10 – 12 Ncm
Zimmer®	Tapered Screw-Vent® 5.7	30 Ncm	10 – 12 Ncm